



## How to create a custom certificate

Resusci Anne Skills Station comes highly configurable and one of the things possible is customization of the completion certificate. Some examples could be adding logos, change text or alter the layout.

Resusci Anne Skills Station can read files of either HTML format or Flash (.SWF). Note that if Flash is opted for then some software, like Adobe Flash Professional, able to generate .swf files is needed.

This document assumes that the reader has experience with HTML and / or Flash programming.

### Examples

Examples of HTML and Flash certificates can be found in the Resusci Anne Skill Station installation folder:

`\config\certificate.htm`

`\config\certifcate fla`

The easiest way to change the certificate in to how it is desired to look, is to open the examples provided and edit them or use them as inspiration.

### Variables

In order for the name, date and unique certificate number to be inserted into the certificate it is important to include the variables on the correct locations in the certificate. Use the below variable names:

Name:                   %%name%%

Date completed:       %%date%%

Certificate Number:   %%certNumber%%

### Using the new certificate

After the certificate has been created it should be copied to the `\config\` folder. Then you must edit the file “manikin.ini”. Find the place where it says `certFile = certificate.swf`

Edit `certificate.swf` to the desired file name.

Next time you start Resusci Anne Skills Station the new certificate will be used. The change is retroactive and old certificates will open in the new format

### Support

Please visit <http://www.laerdal-sophus.dk> for support and contact information